



STAT-Site[®] M^{Hgb}

Power in the Palm
of Your Hand—
Truly Portable
Hemoglobin Testing



STANBIO
LABORATORY

The World's Source For Clinical Diagnostics

CLIA'88
WAIVED

STAT-Site® M^{Hgb} Products

Catalog No. 900900

STAT-Site® M^{Hgb} Meter

Includes STAT-Site® M^{Hgb} meter, operating manual, quick reference guide and battery (extra battery also included)



Catalog No. 901025

STAT-Site® M^{Hgb} Test Cards

Includes 100 test cards (4 packages of 25 each) and a lot-specific CODE key



Catalog No. 900905

STAT-Site® M^{Hgb} Battery Pack

Includes 5 batteries (approximately 5,000 tests)

Catalog No. 503000

STAT-Site® Hemoglobin Controls

6 x 1.5 mL (3 low and 3 high level)

STAT-Site® M^{Hgb} Specs

Methodology _____	Reflectance photometry
Principle _____	Azidemethemoglobin
Calibration _____	Automatic (CODE Key)
Sample Type _____	Whole blood
Sample Size _____	1 drop (approximately 12 µL)
Linearity _____	6 to 21 g / dL
Within Run Precision CV _____	3.5% to 4.9%
Total Precision CV _____	2.9% to 4.2%

Portable.

The **STAT-Site® M^{Hgb}** is a battery operated, completely portable hemoglobin analyzer that fits in the palm of your hand. Finally, point-of-care testing right at your patient's side.

Advanced.

The **STAT-Site® M^{Hgb}** uses state-of-the-art electronics designed for optimum performance and reliability. Self-prompting graphics guide you through the simple test procedure. It even includes a "Last Result" feature that allows you to recall the last hemoglobin result.

Simple.

Hemoglobin testing is quick and easy—reducing personnel training and improving performance. Unlike competing systems, the **STAT-Site® M^{Hgb}** is virtually maintenance-free.

Powerful.

With one single drop of fingerstick blood, the **STAT-Site® M^{Hgb}** system provides precise hemoglobin analysis in seconds. Treatment can begin immediately, improving patient outcomes and lowering costs.

Easy as one, two, three.

Step 1 INSERT THE TEST CARD

Step 2 APPLY THE SAMPLE

Step 3 READ THE RESULT

CE CLIA '88 WAIVED ISO13485:2003 KEMA CERTIFIED

CPT CODE: 85018QW



STANBIO
LABORATORY

The World's Source For Clinical Diagnostics

- General & Special Chemistry
- Pregnancy
- Point-of-Care
- Rapid Tests
- Therapeutic Drug Monitoring
- Serology
- Occult Blood
- Urinalysis
- Clinical Instrumentation

Stanbio Laboratory
1261 North Main Street
Boerne, Texas 78006 USA

866.782.6246 (U.S. Toll Free)
Tele: 830.249.0772
Fax: 830.249.0851

email: stanbio@stanbio.com

www.stanbio.com

DISTRIBUTED BY

Lit#RSS0003.01 ©2004 Stanbio Laboratory. Printed in USA.
U.S. Patent Nos. 5,104,619; 5,270,219 • Canadian Patent No. 1321945 • European Patent Nos. 0395680, 0512059. Additional patents pending; Canadian Patent Application No. 2,067,724; German Patent Application No. P2851635.7; Italian Patent Application No. 0395680.